

# HYCOL™ HYDROLYZED COLLAGEN

## GEL APPLICATION GUIDE

1



Use the gel on dry or lightly to moderately exudative wounds. For initial application, debride the wound bed of slough and necrotic tissue. HYCOL Gel should be applied directly to the wound with an applicator of choice.

2



Apply gel to the wound bed. The amount applied depends on the wound depth and size. Initial application may require more gel than subsequent applications.

3



Be sure to cover the entire area of the wound bed and wound margin.

4



Gently push gel into any areas of undermining.

- Apply an appropriate non-adherent secondary dressing
- Reapply a minimum of 2-3 times a week or as necessary

Product #	Description	Size	Qty/Box	HCPCS
HCG1010	HYCOL™ Hydrolyzed Collagen Gel	10 g	10	A6011

Manufactured in the USA for Wound Care Innovations, LLC., a Sanara MedTech Inc. Company.

HYCOL is a trademark of Wound Care Innovations, LLC., a Sanara MedTech Inc. Company.

© 2020 Sanara MedTech Inc. All rights reserved. Rev. 200224



**Sanara**  
MedTech  
Evidence Based Healing

# HYCOL™ HYDROLYZED COLLAGEN

## POWDER APPLICATION GUIDE

1



Use the powder on moderately to heavily exudative wounds. For initial application, debride the wound bed of slough and necrotic tissue. HYCOL™ Powder can be applied in targeted amounts using the puffer tip. The tip can also be removed for pouring powder into the wound.

2



Apply powder to the wound bed. The amount applied depends on the wound depth, size and amount of exudate. Initial application may require more powder than subsequent applications.

3



Be sure to cover the entire area of the wound bed and wound margin.

4



Gently push powder into any areas of undermining.

- Apply a secondary dressing appropriate for the level of drainage
- Reapply a minimum of 2-3 times a week or as necessary until exudate is controlled

Product #	Description	Size	Qty/Box	HCPCS
HCP0110	HYCOL™ Hydrolyzed Collagen Powder	1 g	10	A6010

Manufactured in the USA for Wound Care Innovations, LLC., a Sanara MedTech Inc. Company.

HYCOL is a trademark of Wound Care Innovations, LLC., a Sanara MedTech Inc. Company.

© 2019 Sanara MedTech Inc. All rights reserved. Rev. 191022

