


Cellerate[®]_{RX}

ACTIVATED COLLAGEN[®]

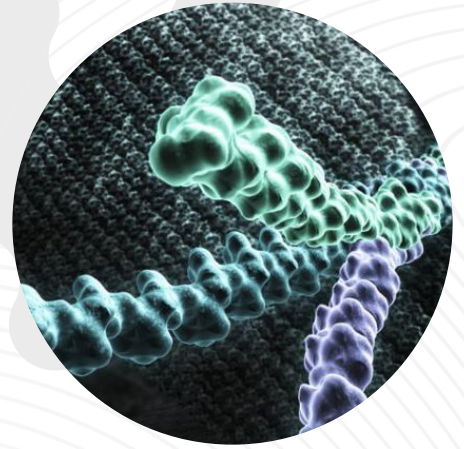

THE HYDROLYZED COLLAGEN DIFFERENCE

SAFE. EFFECTIVE. VERSATILE.


CellerateRX[®] Hydrolyzed Collagen is Type I bovine collagen and contains no additives. It is indicated for:

- Chronic and acute wounds
- Partial- and full thickness wounds
- Pressure injuries I-IV
- Venous stasis ulcers
- Arterial ulcers
- Diabetic Ulcers
- Traumatic wounds
- 1st and 2nd-degree burns
- Superficial wounds

KEY FEATURES

- CellerateRX[®] Hydrolyzed Collagen provides the benefit of collagen fragments that are approximately 1/100th the size of native collagen
- The body does not have to break it down before use
- Provides a moist wound environment conducive to wound healing
- Application process is simple
- May be used with many dressing types
- Contains no additives
- Three-year shelf life

CellerateRX[®] Hydrolyzed Collagen is approximately 1/100th the size of native collagen.

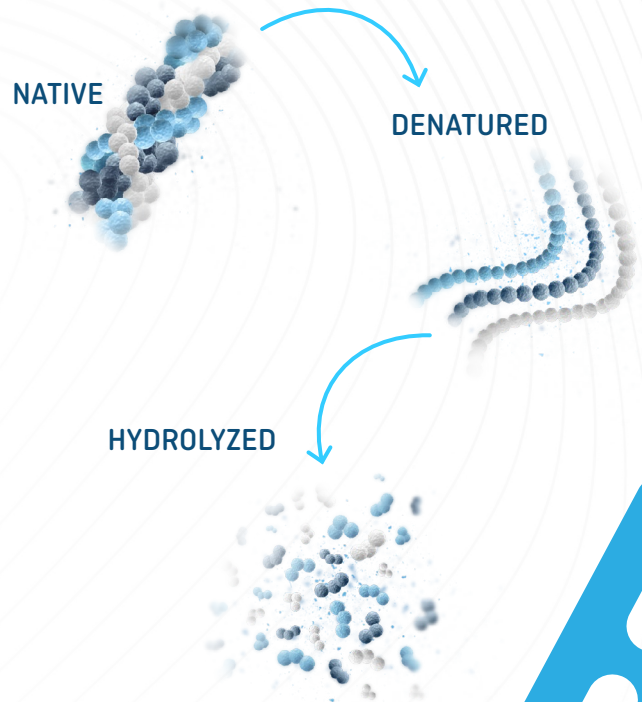

With CellerateRX[®] Hydrolyzed Collagen, the body does not have to break it down, so the benefits of collagen are provided to the wound.



BREAKING DOWN THE BENEFITS



Effective

Provides fragments to the wound bed that are a fraction of the size of native collagen




Versatile

Conforms to any wound bed, including tunneling, and undermining wounds



Easy to Apply

Can be used on a variety of wounds

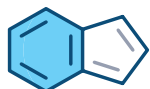

Compatible


May be used with appropriate secondary dressing

APPLICATION

1

Use the gel on dry or lightly to moderately exuding wounds. For initial application, debride the wound bed of slough and necrotic tissue.

CellerateRX Gel should be applied directly to the wound with an applicator of choice.

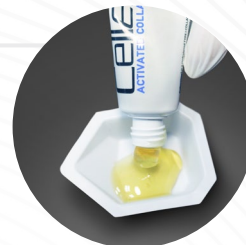


2

Apply gel to the wound bed. The amount applied depends on the wound depth and size. Initial application may require more gel than subsequent applications.




3

Be sure to cover the entire area of the wound bed and wound margin.

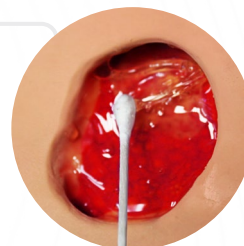

4

Gently push gel into any areas of undermining.

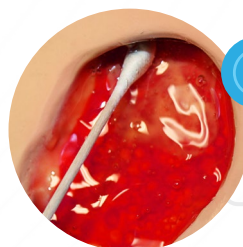

- Apply an appropriate non-adherent secondary dressing
- Reapply a minimum of 2-3 times a week or as necessary

Cellerate[®] RX
ACTIVATED COLLAGEN[®]



For more information call
800-205-7719

Visit sanaramedtech.com

Manufactured in the USA for Wound Care Innovations, LLC., a Sanara MedTech Inc. Company

CellerateRX is a registered trademark of Applied Nutritionals, LLC. Activated Collagen is a registered trademark of Wound Care Innovations, LLC., a Sanara MedTech Inc. Company. © 2023 Sanara MedTech Inc. All rights reserved. MKT.01.19 Rev. 230303

Ordering Information

Product Code	Product Description	Size	HCPCS Code
CRXG-28GMEA	CellerateRX [®] Gel	28gm	A6011