

## GEL APPLICATION GUIDE

1



Use the gel on dry or lightly to moderately exudative wounds. For initial application, debride the wound bed of slough and necrotic tissue. HYCOL<sup>®</sup> Gel should be applied directly to the wound with an applicator of choice.

2



Apply gel to the wound bed. The amount applied depends on the wound depth and size. Initial application may require more gel than subsequent applications.

3



Be sure to cover the entire area of the wound bed and wound margin.

4



Gently push gel into any areas of undermining.

- Apply an appropriate non-adherent secondary dressing
- Reapply a minimum of 2-3 times a week or as necessary

Product #	Description	Size	Qty/Box	HCPCS
HCG1010	HYCOL <sup>®</sup> Hydrolyzed Collagen Gel	10 g	10	A6011

Manufactured in the USA for Wound Care Innovations, LLC., a Sanara MedTech Inc. Company.

HYCOL is a registered trademark of Wound Care Innovations, LLC., a Sanara MedTech Inc. Company.

© 2021 Sanara MedTech Inc. All rights reserved. Rev. 210712



**Sanara**  
MedTech  
Evidence Based Healing

## POWDER APPLICATION GUIDE

1



Use the powder on moderately to heavily exudative wounds. For initial application, debride the wound bed of slough and necrotic tissue. HYCOL<sup>®</sup> Powder can be applied in targeted amounts using the puffer tip. The tip can also be removed for pouring powder into the wound.

2



Apply powder to the wound bed. The amount applied depends on the wound depth, size and amount of exudate. Initial application may require more powder than subsequent applications.

3



Be sure to cover the entire area of the wound bed and wound margin.

4



Gently push powder into any areas of undermining.

- Apply a secondary dressing appropriate for the level of drainage
- Reapply a minimum of 2-3 times a week or as necessary until exudate is controlled

Product #	Description	Size	Qty/Box	HCPCS
HCP0110	HYCOL <sup>®</sup> Hydrolyzed Collagen Powder	1 g	10	A6010
HCP0512	HYCOL <sup>®</sup> Hydrolyzed Collagen Powder	5 g	12	A6010

Manufactured in the USA for Wound Care Innovations, LLC., a Sanara MedTech Inc. Company.

HYCOL is a registered trademark of Wound Care Innovations, LLC., a Sanara MedTech Inc. Company.

© 2021 Sanara MedTech Inc. All rights reserved. Rev. 210712



**Sanara**  
MedTech  
Evidence Based Healing